ANLEITUNG

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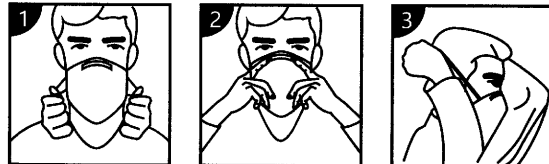
USAGE OF MASK -02 :

STORAGE CONDITIONS AND METHODS:

The storage temperature is -20~38°C, and the relative humidity is not more than 80%. Well ventilated and clean environment without corrosive gas, please do not use if the product gets mildewed or deteriorated due to improper storage.

FFP2 : EN149:2001 + A1:2009

MANUFACTURER:

Hunan Dreaming Cloud E-Commerce CO., Ltd
ADDRESS: Block 1, Smart Tech Park, 57# Huangxing Avenue, Changsha Economic and Technological Development Zone, Changsha, Hunan, China Made in China

INSTRUCTIONS**Standard:**

This product meets the EN 149:2001+A1:2009 standard for respiratory protective devices - Filtering half masks to protect against particles. This filtering face mask is classified as category III Personal Protective Equipment (PPE) impressed EC marking in conformity with European Commission Regulation (EU) 2016/425 relating to PPE.

Intended use:

The dust mask is classified as category FFP 2. It provides protection against particulates, mist, smoke and oil -based aerosols. The packaging provides the mask with hygienic protection prior to its use. Can effectively protect pollen allergy, etc. The mask can only be used for personal protection.

Warnings:

The user must be trained and given the correct instructions about how to wear the mask. FFP masks do not provide protection against gases and vapours.

The ambient oxygen concentration may not be less than 19.5 Vol.%.

The masks should not be deployed if the concentration, type or nature of the contamination is unknown or is of a hazardous level.

Masks must be replaced after they have been damaged, if the breathing resistance is increased as a result of clogging, and after use.

No changes may be made to the mask.

Instructions:

The mask is intended to be used only once. Do not use more than once.

Keep the packaging closed until you will use the mask.

Replace the mask if it is damaged or if the breathing resistance increases due to clogging.

Not to be used during firefighting activities.

Not to be used in an explosive atmosphere.

Non-observance of the instructions and warnings for using this mask could lead to a poorer performance of the mask and could result in health problems.

Keep facial hair from underneath the mask. The mask cannot offer maximum protection when it fails to properly connect to your face due to facial hair.

Transport and storage:

Do not expose to direct sunlight. Store in a dry and frost-free location.

Usage:

1. Take the mask in one hand and hold the nose piece between your fingertips. Ensure that the ear straps are hanging freely without making contact with your hand.

2. Press the mask against your chin, with the nose piece pointing towards the front. Keep facial hair from between the mask.

3. Face the mask without the nose clip so that the nose clip is above the mask.

4. Pull the ear strap behind the ear and hang it.

5. Press the nose clip to both sides with both hands to make the nose clip fit to the tightness of the mask.

6. Try the mask before you enter the work area.

For detailed steps, please refer to the outer packaging.

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